medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T +32 011 24 25 60
F +32 011 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Brasil
Rua Neusa 216
Diadema-Sao Paulo
Cep 09941-640
Brazil
T +55 11 3201 1188
F +55 11 3201 1185
sac@medibrasil.com
www.medibrasil.com

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbæk Strand
Denmark
T +45 46 55 75 69
F +45 70 25 56 20
kundeservice@sw.dk
www.medit danmark.dk

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi-hungary.hu
www.medi-hungary.hu

medi Medical Support Sdn Bhd
medi representative office Asia
Unit No. B-2-19, Block B, No.2,
Jalan PJU 1A/7A
Oasis Ara Damansara, PJU 1A,
47301 PETALING JAYA
Selangor Darul Ehsan
Malaysia
T: +6 03 7832 3591
F: +6 03 7832 3921
info@medi-asia.com
www.medi-asia.com

medi Middle East
P.O. Box: 109307
Abu Dhabi
United Arab Emirates
T +971 2 6429201
F +971 2 6429070
info@mediuae.ae
www.mediuae.ae

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi Polska Sp. z o. o.
Zygmunta Starego 26
44-100 Gliwice
Poland
T+48 32 230 60 21
F+48 32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
Rua Eugénio de Castro Rodrigues, nº2
1700-184 Lisboa
Portugal
T +351 21 843 71 60
F +351 21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LRC
Business Park "Rumyantsevo"
Rumyantsevo, Build. 1
Leninsky Township
142784 Moscow Region
Russia
T +7 495 229 04 58
F +7 495 229 04 58
info@medirus.ru
www.medirus.ru

medi UK Ltd.
Plough Lane
Hereford HR4 OEL
United Kingdom
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi Ukraine LLC
Tankova Str. 8, office 35
Business-center "Flora Park"
Kiev 04112
Ukraine
T +380 44 591 1163
F +380 44 455 6181
office@medi.ua
www.medi.ua

medi USA L.P.
6481 Franz Warner Parkway
Whitsett, N.C. 27377-3000
USA
T +1 336 4 49 44 40
F +1 888 5 70 45 54
orders@mediusa.com
www.mediusa.com

medi

medi orthocox

Hüftrotationsbandage
Hip rotation support
Bandage anti-rotation de la hanche
Vendaje para el giro de la cadera

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medi orthocox

Hip rotation support

The circular compression on the hip and thigh ensures the required compression of the operation site and supports the direct postoperative positioning of the treated hip joint. At the same time, the elastic tension on the thigh counteracts the usual external rotation of the leg.

The compact design of the hip rotation support limits involuntary and uncontrolled movements, and offers the patient the additional sense of security that is necessary for rapid and wide-ranging mobilisation.

The use of medi orthocox simplifies compression treatment of the leg and pelvis in hospital and during rehabilitation and thus makes a lasting contribution to the treatment results with better economy.

Indications

For combined compression and adjustment of external rotation after

- Intertrochanteric correction osteotomy
- Osteosynthesis for pertrochanteric fractures
- Total endoprosthesis of the hip

Relative contraindications

- Arterial circulation disorders
- Massive leg deformities
- Pre-existing gangrenous damage

Important notes

Due to the compression effect on the trochanters, particular attention must be paid to the arterial blood supply to this area.

Materials

67 % polyamide
25 % polyester
8 % Elasthane
latex-free

Fitting instructions

- medi orthocox is worn around the hips and the thigh (fig. 1).
- For hip compression pass the upper free end around your pelvis. Adjust the tension to vary the compression to suit the patient (fig. 2).
- Pass the lower free end around the proximal part of the thigh. The elastic tension of the material exerts pressure to reduce external rotation of the leg (fig. 3).
- The free ends of the support are secured by broad Velcro fasteners, thus enabling the patient to adjust compression and thigh rotation independently (fig. 4).

Care instructions

Close all the Velcro fasteners before washing to maintain their function and avoid damage to other articles. Wash the support in a normal washing machine with a non-bleaching detergent. Do not use any softener.

**Storage instructions**

Keep the product in a cool, dry place and do not expose to direct sunlight.

Warranty / contractual guarantee

Consistent with legal guidelines

Accessories

Pockets for ice compresses
medi orthocox supplies optional pockets for cooling the operation site with ice compresses (28 cm x 12 cm), which can be fastened to the support with Velcro.

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